*Oggetto:*

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*Subject: Products for hygiene and disinfection of dental surgeries*

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## ECO-JET 1 spray

**DISINFETTANT 500 ml**

It is particularly recommended for the routine cleaning and disinfection of cracked, rough or difficult to reach surfaces of medical devices (operating lamp and chest of drawers handles, hand pieces, trays, chair, and so on...). Ready to use: spray onto the

surface from 20/30 cm, let it work for 5 minutes, then rub with a dry sterile cloth.

### Composition:

100 g. of active solution contain: Propylglycol n-Butyl Ether 0,6 g, Didecyl Dimethyl Ammonium Chloride 0,56 g, ethano and isopropyl alcohol 10%, perfume and excipients that are necessary to 100 g.

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### Disinfecting wet wipes

(Box of 100 tissues)

They are particularly recommended for the routine cleaning and disinfection of small surfaces and objects of medical devices (chair armrest, operating lamp and chest of drawers handles, hand pieces, shelves, trays, and so on...).

Ready to use: rub the surface to be treated to clean dirt away, let it work for 5 minutes. Self-drying.

#### Features

Wipes in non-woven polypropylene tissue. Does not leave fibres. Cool, flower-scented perfume.

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## LUBRI-JET

Spray lubrificante 150 ml

LUBRI-JET is a lubricant suitable for dental aspirators; it is particularly fit to keep rubber and plastic soft and flexible. Recommended for rubbing surfaces, rubber, O-ring, plastic materials and metals. Especially suitable for lubricating articulations of clamps and scissors before placing them in autoclave. Contains very pure substances; propellant is not flammable and is dust-repulsive.

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If you do not have the "Guide to dental surgery disinfection" in your own language, please ask for it to CATTANI S.p.A.

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Company with Quality System Certified  
by DNV UNI EN ISO 90012000

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**also for these reasons  
.. we have chosen CATTANI**

CATTANI S.p.A.

VIA NATTA, 6/A - 43100 PARMA - ITALY

TEL: +39 0521 607604 - SALE DEPT. FAX: +39 0521 607628 - PURCHASING DEPT. FAX: +39 0521 607855 - ACCOUNTING DEPT. FAX: +39 0521 399966  
<http://www.cattani.it> Email: [cattani@in.it](mailto:cattani@in.it)

Company with Quality System Certified by DNV UNI EN ISO 9001/2000  
- UNI CEI EN ISO 13485 -

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Si non tienen la guía de la desinfección del estudio dentista en su idioma, pueden pedirla a CATTANI S.p.A.